

EyePro Eyelid Cover is intended to aid in keeping the eye closed during medical procedures. It is suitable for patients undergoing anaesthesia or patients being monitored in an unconscious state.

#### INSTRUCTIONS FOR USE

- Ensure eye is clean and dry before application of EyePro Eyelid Cover.
- When applying the dressing, peel backing off and place over the closed eye.
- Do not apply pressure to the clear central window. This will increase the risk of eyelid trauma and/or eyelash injury on removal.
- Make sure the closed eye is visible through the transparent, central window.
- Apply gentle finger pressure around the outer white section to maintain a good seal around the eye.
- Remove slowly and gently using side tabs.
- Dispose of appropriately.

#### WARNINGS

- **Product is designed to keep eye closed. It is not intended to protect the eye from fluids.**
- If fluids permeate the white adhesive fabric and begin to pool in the eyelid area, immediately remove, clean and dry the area, then replace with new EyePro Eyelid Cover.
- Use with caution on patients who are debilitated, atopic, have decreased or damaged skin integrity or have any other condition where their skin or eyelashes may be more easily damaged

