



Type 2 Process Indicator



7kg Mimic



Lead Free

The Interster Bowie & Dick Autoclave Lean Pack aligns with the requirements of AS4187 as a steam penetration, air removal test. The pack consists of a chemical indicator which is packed in between layers of special paper. The chemical indicator and special paper are packed in a cardboard outer packaging which can be opened very easily after exposure. The product does not contain heavy metals. Result record sheet included.

DISTINCT COLOUR CHANGE

From blue to pink

INDICATOR INK TECHNOLOGY

Silkscreen printing process can detect steam quality issues

HEAVY METAL FREE

Eliminates concerns over exposure to lead or heavy metals



Suitable for:


Autoclave sterilisers

Reorder	Feature	Sterility	Pcs/Unit	Units/Ctn
3FSKS610860	BSI Kitemark	Non-Sterile	20	1


Test results Bowie & Dick Type Autoclave Lean Pack:




Art. no. 3FSKS610860 - 3,5 min.

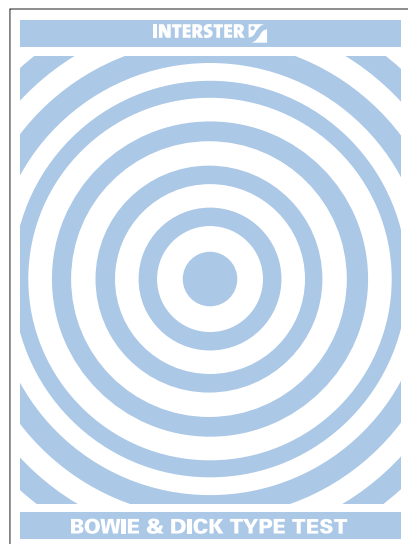


Before use

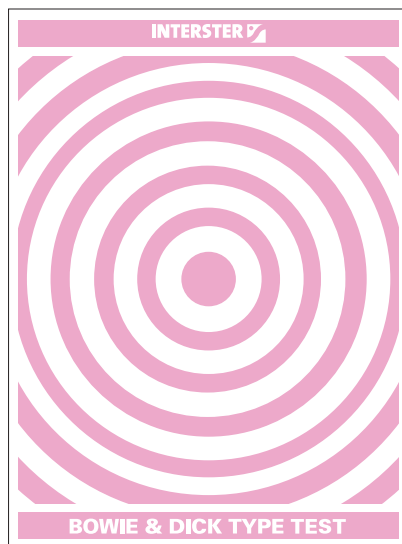


Completely changed sheet
Minimum even colour change that must be achieved.

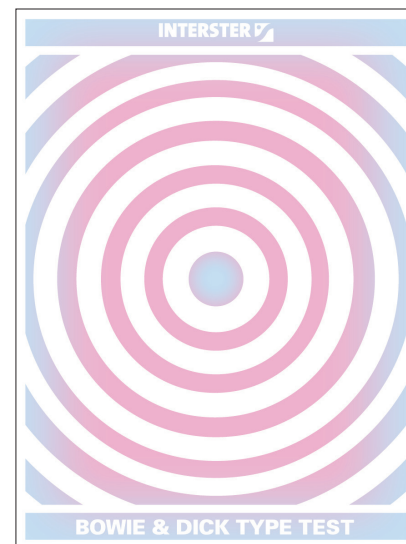


Not properly changed sheet
Possible causes: a too short sterilization time or too low temperature.

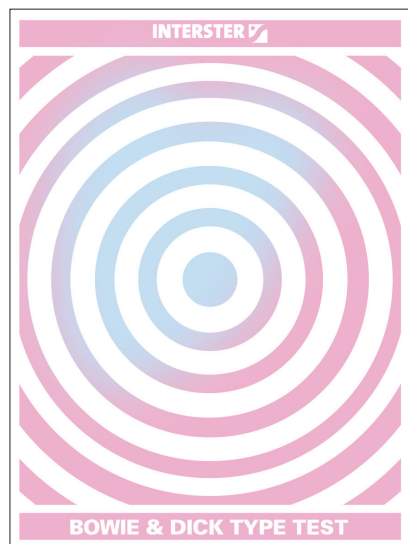

Not properly changed sheet
A clearly defined mark is visible.
Explanation: presence of air or non-condensable gases. Possible causes: poor air removal, leakage or gas in the steam and sub-atmospheric pulsing.

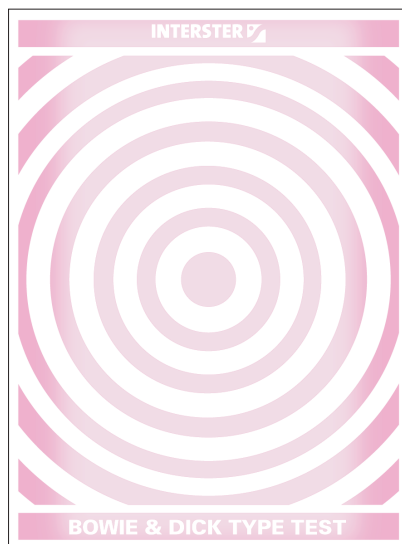

Not properly changed sheet
An indistinct mark, almost always in the middle, is visible.
Explanation: superheated steam. Possible causes: the test pack has been placed in the sterilizer without immediately starting the process (desiccation), too high steam velocities or steam supply pressure.

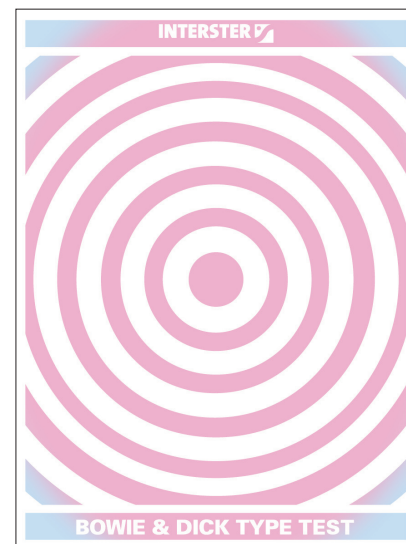

Not properly changed sheet
Completely changed sheet except for the corners.
Possible causes: wet steam.

Do not use the above shown pictures as a reference colour