

ISB403, ISB404 - IV Dressing Reinforced (*Instructions For Use*)

The Asguard IV Dressing Reinforced are constructed from a High MVTR film, which allows for excess moisture to evaporate, combined with a laminated non-woven border for securement and enhanced fixation. They maintain firm adhesion and allow for insertion site monitoring.



STEP 1

Prepare site as per local protocol and ensure skin is clean and dry before application.
Remove backing to uncover adhesive side of dressing.



STEP 2

While positioning the notch over the catheter, position clear window over the insertion site.
Apply gentle pressure across the surface of the dressing, ensuring adherence.



STEP 3

Smooth edges of dressing whilst gently removing the paper frame.
Ensure thorough adhesion by smoothing down dressing from centre to outer edges.



STEP 4

Apply securement tape(s) and insertion date label as preferred and required by local protocol.



REMOVAL

To minimise potential damage to fragile skin, Asguard IV Dressing Reinforced should be eased off using the "stretch and release" technique or with an adhesive remover wipe.

INDICATIONS

Asguard IV Dressing Reinforced are used to dress IV insertion sites and to provide primary fixation of intravenous access devices.

CONTRAINDICATIONS

Should not be used if allergies to any of its components is known.

At Sentry, we are dedicated to upholding sustainable principles to safeguard our planet, people, and future, and we strive to consistently act responsibly in doing so.

Sentry Medical Pty Ltd

1300 995 999 | sales@sentrymedical.com.au
www.sentrymedical.com.au

