



STEP 1

Insert I.V. cannula using local protocols. The skin surrounding the wound should be clean and dry.

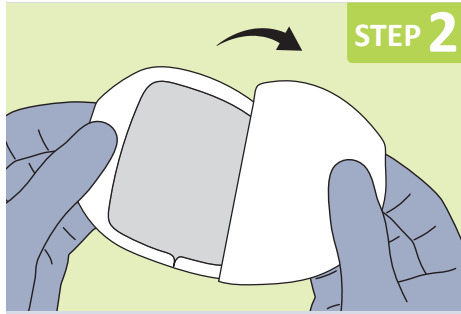


STEP 2

Open the sterile packaging and remove the dressing. Separate the paper backing from the AsGuard Clear Film Dressing.

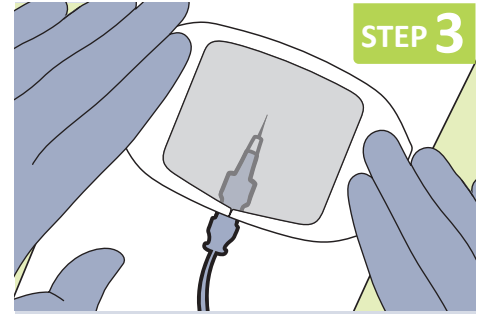

STEP 3

Apply the AsGuard Clear Film Dressing covering insertion site and I.V. cannula hub. Apply gentle pressure around the edge of dressing.

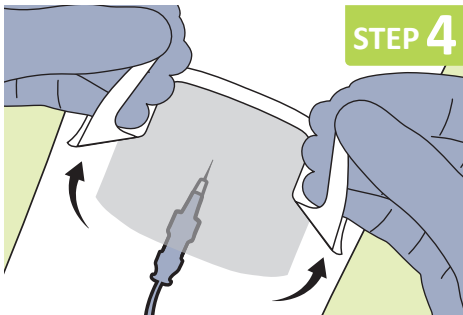

STEP 4

Peel the window frame away from the dressing. Smooth down the edges of the clear border to create a waterproof and secure seal. Complete and apply dating label as preferred and required by local protocol.


INDICATIONS

AsGuard Clear Film Dressings are indicated for use as dressings in the management of all types of insertion sites such as peripheral IV cannula or used as a fixation layer to provide a waterproof environment and/or to keep non-adhesive primary dressings in place.

CONTRAINDICATIONS

Should not be used if allergies to any of its components, is known.

DRESSING CHANGE

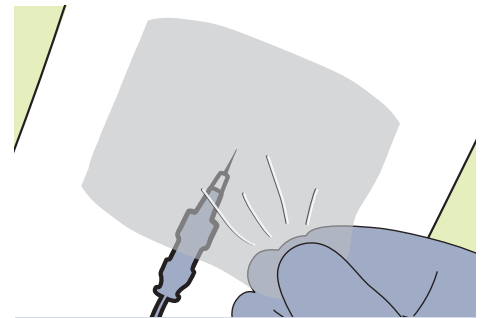

7 days is the maximum wear time for dressing.

To minimise potential damage to fragile skin, AsGuard Clear Film Dressing should be eased off, using a "stretch and release" technique or an adhesive remover wipe.