



STEP 1

Cleanse wound thoroughly, using local protocols. The skin surrounding the wound should be clean and dry. As AsGuard Clear film is a fixation dressing only, a dressing of choice should be used to cover the wound.

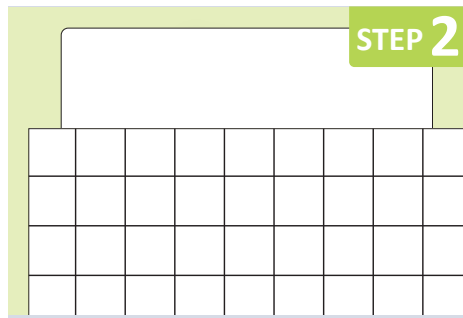

STEP 2

Film size is dependent on the dressing surface area. Use grid to determine length – the film edges should extend 2-3cms beyond the dressing edges.

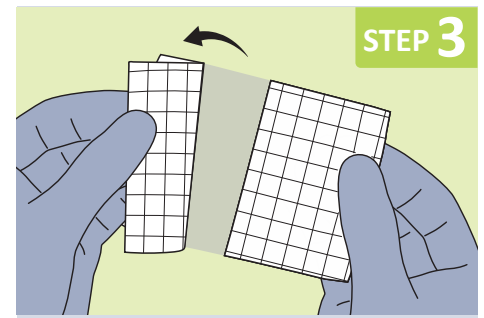


STEP 3

Cut to desired length. Holding paper backing tab, separate the backing (marked) from the AsGuard Clear Film.



STEP 4


Apply the AsGuard Clear Film Roll covering entire dressing. Apply gentle pressure around the edge of film.

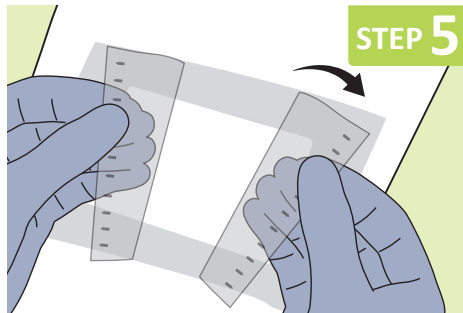

STEP 5

Peel the support layer (labelled '3') away from the film. Smooth down the edges of the clear border to create a waterproof and secure seal.

DRESSING CHANGE

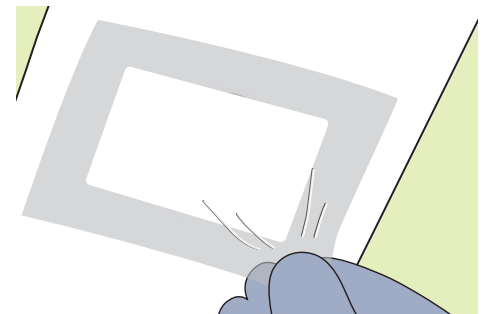

The clinician-in-charge is responsible for determining the need for dressing changes, dependent on the stage and phase of wound healing and exudate level.

7 days is the maximum period between dressing changes.

To minimise potential damage to fragile skin, AsGuard Clear Film Roll should be eased off, using a “stretch and release” technique or an adhesive remover wipe.

INDICATIONS

AsGuard Clear Film Rolls are non sterile and indicated for use as a fixation layer to provide a waterproof environment and/or to keep non-adhesive primary dressings in place over:

- Minor skin tears
- Abrasions and lacerations
- Blisters and minor skin loss

CONTRAINDICATIONS

Should not be used if allergies to any of its components, is known.