



STEP 1

The skin where application will take place should be clean and dry. As AsGuard Flex is a fixation dressing only, a dressing of choice should be used to cover any broken skin.

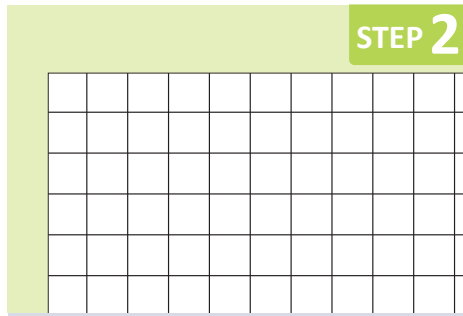


STEP 2

AsGuard Flex Non-woven size is dependent on the device being secured. Use grid to determine length – the Non-woven adhesive should extend 2-3cms beyond the device edges.

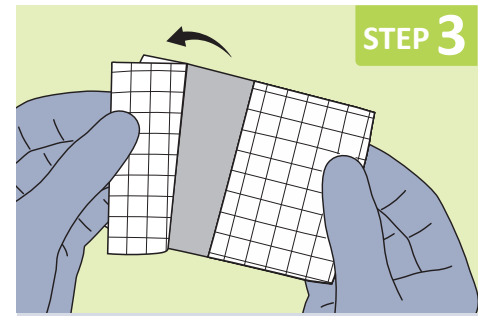


STEP 3

Cut to desired length. Remove backing paper from the AsGuard Flex.

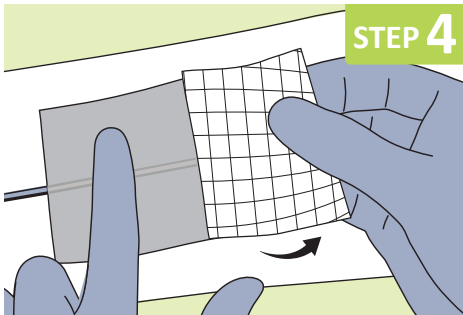

STEP 4


Apply the AsGuard Flex Non-woven dressing covering the device being secured. Release backing paper and apply over device.

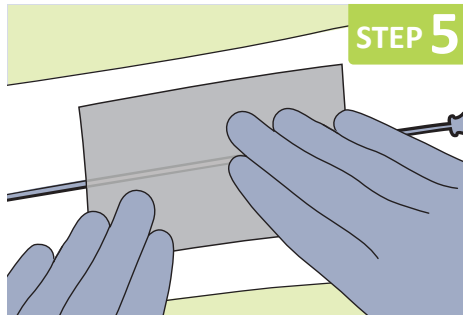

STEP 5

Smooth down the Non-woven adhesive from the centre to the edges, ensuring adherence.

DRESSING REMOVAL

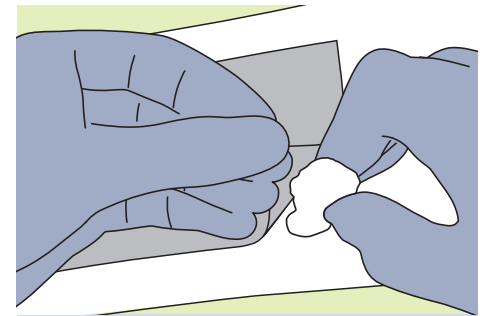

7 days is the maximum period AsGuard Flex should be left in situ.

To minimise potential damage to fragile skin, AsGuard Flex Non-woven Roll should be eased off using an adhesive remover wipe.

INDICATIONS

AsGuard Flex Non-woven Rolls are non-sterile and indicated for use as a fixation layer to keep non-adhesive primary dressings and other medical devices in place.

CONTRAINDICATIONS

Should not be used if allergies to any of its components, is known.