


Pill Crusher (*Instructions For Use*)

Senturian's Pill Crusher and Pouches are the two parts of a complete integral system to quickly crush tablets into easy-to-consume powder. Only Senturian Pouches should be used with the Senturian Crusher. The use of any other container and/or any misuse of the Senturian Crusher can void the Limited Warranty. In addition, other containers may be punctured during use, resulting in loss of medication.




1. Remove a Senturian Pouch from the handy built-in dispenser. Place pill(s) into pouch. Depending on the size of the pills, several pills can be crushed at one time within the same pouch.

NOTE: Please consult your pharmacist or doctor before mixing or crushing any medications.

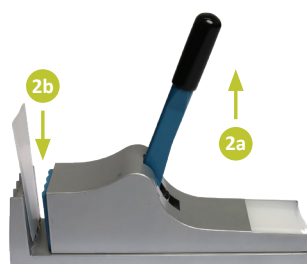

2. Lift handle to open crushing area (2a) and insert filled pouch into the open area (2b).
3. Press handle down to crush. Once is sufficient to break pills into a coarse powder. Repeat for finer powder. Better results may be achieved with a small shake of the pouch after the first press.

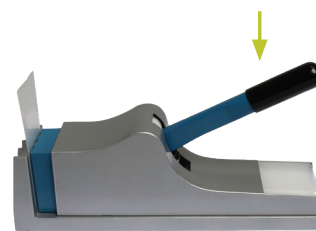

Once the desired particle size is achieved, remove the pouch and administer medication. Tap the sides, bottom and corners to retrieve any residual powder.

CLEANING & MAINTENANCE

For regular maintenance, Senturian Pill Crusher can be cleaned with a damp cloth.

CAUTION:

Keep hands and fingers out of the crushing area to prevent accidental injury.

IMPORTANT:

1. Check with your pharmacist or doctor before crushing medications.
2. Certain medications should not be crushed, such as enteric coated pills and sublingual tablets.
3. Please ensure that the medication can be taken with food if you intend to mix it into food or beverage.
4. Consult your pharmacist or doctor before mixing medications.

Limited Warranty

Manufacturer warrants that the **Senturian Crusher** is free of all defects and functional failure for the useful life of the product. This warranty is void if (1) any container other than **Senturian Pouches** have been used to crush tablets with the **Senturian Crusher**; (2) **Senturian Crusher** has been used to crush anything other than common tablets/pills; (3) **Senturian Crusher** has been damaged by misuse, misapplication or accident. To make a warranty claim, please return product, freight prepaid to Sentry Medical Pty Ltd.

At Sentry, we are dedicated to upholding sustainable principles to safeguard our planet, people, and future, and we strive to consistently act responsibly in doing so.