



STEP 1

Cleanse wound thoroughly, using local protocols. The skin surrounding the wound should be clean and dry.



STEP 2

Apply Zorflex, either side down. Apply with the direction of the stretch running along the limb, to allow for movement



STEP 3

Gently ensure good contact with the surface of the wound bed.



STEP 4

Cover with secondary dressing appropriate to the level of exudate, or apply NPWT (as per manufacturer's guidelines).



STEP 5

Secure with second dressing with product of choice
Tape: adhesive or silicone
Bandage: flat or tubular.

For full IFU, refer to instructions provided with product.

GENERAL INFORMATION

- Secure the Zorflex dressing in place using an appropriate secondary dressing or compression therapy.
- Zorflex allows transfer of the exudate into the secondary dressing which should be replaced as necessary without disturbing the Zorflex or the wound bed.
- Very low levels of exudate may cause adherence. When this condition exists Zorflex LA (Low Adherence) should be used.

INDICATIONS

Zorflex is indicated as an antimicrobial dressing over discharging, partial and full thickness wounds such as:

- Surgical sites
- Fungating carcinomas
- Pressure ulcers
- Venous ulcers
- Diabetic ulcers
- Donor and recipient graft sites



On removal of secondary dressing Zorflex® surface looks **very wet and shiny** (any wear time up to 7 days): Zorflex® needs changing, remove and replace.



On removal of secondary dressing Zorflex® surface looks **neither dry nor very wet** (wear time less than 7 days): Remove Zorflex® that comes off easily, and replace. Leave any “adhered” product in-situ until 7 days has lapsed, then change.



On removal of secondary dressing Zorflex® surface looks **dry and dull** (wear time less than 7 days): Leave Zorflex® in situ until 7 days has lapsed, then change.

DRESSING CHANGE

If on removal of secondary dressing Zorflex surface looks very wet and shiny (any wear time up to 7 days) - Zorflex needs changing, remove and replace.

If on removal of the secondary dressing Zorflex surface looks neither dry nor very wet (wear time less than 7 days) - remove Zorflex that comes off easily, and replace. Leave any adhered product in-situ until 7 days has lapsed then change.

If on removal of secondary dressing Zorflex surface looks dry and dull (wear time less than 7 days) - leave Zorflex in-situ until 7 days has lapsed then change.