

SENTURIAN[®]

ALL IN ONE

EASY TO USE

FIRST RESPONSE



SKIN TEAR PACK
FIRST RESPONSE

SENTRY
MEDICAL

Senturian Skin Tear Pack

Our Senturian First Response Skin Tear Pack (STP) is an all in one sterile pack designed to enhance the immediate assessment and treatment of skin tears.

With easy to follow instruction guides, the Skin Tear Pack provides all tools required to assess and treat skin tears, ensuring optimal healing and thorough documentation.



Cost Savings for your Facility¹

Average Days to Heal Type 3 Skin Tears



Senturian First Response Skin Tear Packs can provide benefits to facilities through their potential to reduce days to heal

Facility Expenditure Per Type 3 Skin Tear

SENTURIAN STP USE	AVG. DAYS TO HEAL	DRESSING CHANGES	TOTAL COST
Pre STP	51 Days	17 Changes	\$816
Using STP	10 Days	4 Changes	\$148

81% MORE COST EFFECTIVE

1. Based on an independent clinical study performed over a four month period - two months pre Senturian STP and two months using Senturian STP in four Royal Freemasons Aged Care facilities. Comparing treatment of a resident with a skin tear would routinely occur as per Royal Freemasons' existing skin tear guidelines last updated in 2020 vs introducing the Senturian First Response Skin Tear Pack. Days have been rounded to nearest full day. Data on File.



Ideal for First Response



Quick & Easy to Use



Efficient Documentation


Simple to Use

The Senturian First Response Skin Tear Pack provides easy to follow documentation, ensuring thorough treatment and action plans are quick to initiate and record.

Pictorial International Skin Tear Advisory Panel (ISTAP) classification

Clear indication on dressings required

Simple treatment flow chart

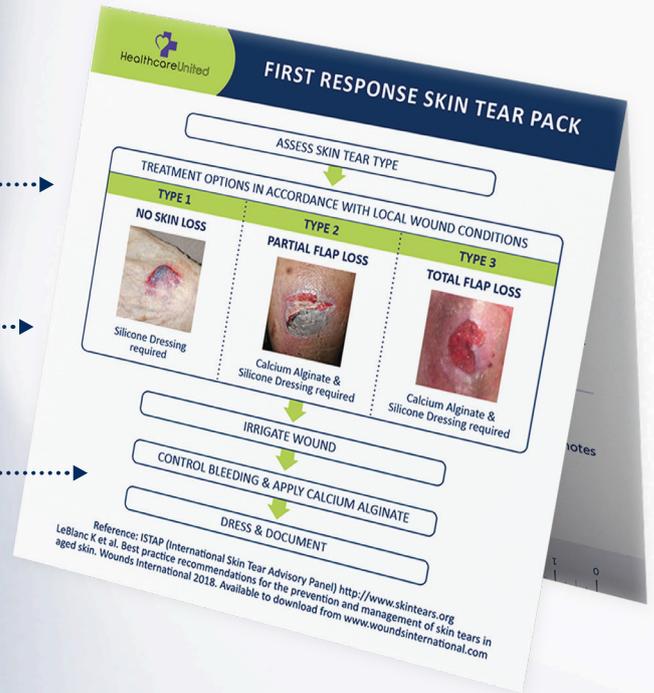

Clear and simple form

Visual on wound location

Protocol checklist

Ruler for wound measurement

WOUND DRESSING PACK

To reduce the risk of infection


QUICK AND EASY TO USE

Suitable for the novice to the most experienced health care practitioner




OSMOCEL CALCIUM ALGINATE

As the primary dressing for its coagulation properties

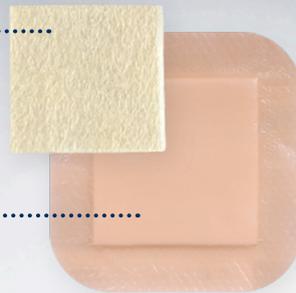

OSMOCEL SI SILICONE FOAM DRESSING

For protection and non-adherent properties to reduce further trauma to the skin

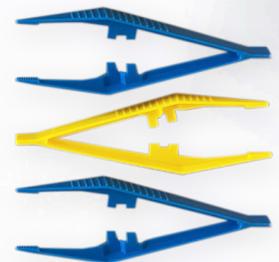

Senturian Skin Tear Packs include different Osmocel Si dressing sizes to meet patient needs

Reorder	Sterility	Pcs/Unit	Units/Ctn
STP7575	Sterile	40	4
STP100100	Sterile	40	4



Skin Tear Pack Contents:

- 1 x PE/Cellulose Sterile Field, Green (50cm x 50cm)
- 1 x White Dressing Tray plain (140mm x 130mm x 18mm)
- 5 x Non Woven Balls
- 1 x Dressing Towel (40cm x 40cm)
- 2 x Non Woven Swabs, 4 Ply (7.5cm x 7.5cm)
- 3 x Disposable Plastic Forceps: 2 x Blue, 1 x Yellow (11cm)
- 1 x AWC001A - Osmonate Calcium Alginate Dressing (5cm x 5cm)
- 1 x Patient Record Form - including ISTAP Skin Tear Classification Reference
- STP7575 only** 1 x AWS003 - Osmocel Si Silicone Foam (7.5cm x 7.5cm)
- STP100100 only** 1 x AWS004 - Osmocel Si Silicone Foam (10cm x 10cm)