

Flamigel® LIVE NON-STOP



1st degree
burn



Sunburn



Post laser
therapy



Minor
scrape



CE 0344

**Your trusted solution for
minor wounds and skin repair**

Sentry
MEDICAL

What is Flamigel®?

Flamigel® is a versatile smart gel specially formulated to aid in the healing of a range on superficial wounds and damaged skin.

Key Benefits:

- Up to 40% faster wound healing¹
- Reduces risk of scarring
- Soothes pain with a cooling effect
- Creates a protective barrier against external contamination
- Hydrates dry wounds and absorbs excess fluid

All year-round, Flamigel® is the go-to solution for your daily wound care needs.

Minor scrape



Sunburn



1st degree burn




Post laser therapy



How does Flamigel® work?

Flamigel® is a hydro-active colloid gel with arginine, containing 4 key ingredients²:

1

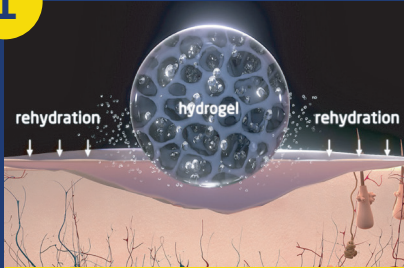


Hydrates dry wounds²

Hydrogel

Moisturises and debrides the wound

2

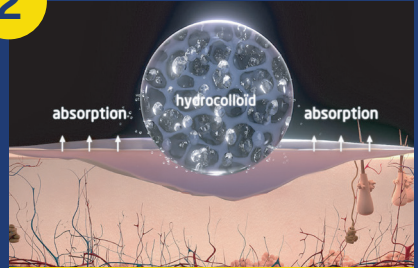


Absorbs excess fluid²

Hydrocolloid

Absorbs excessive fluid and restores a normal level of hydration to the wound

3

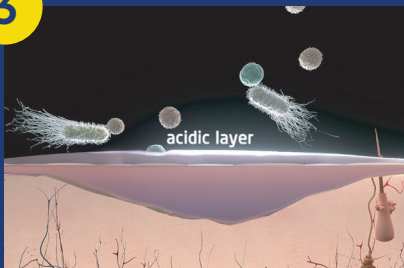


Creates bacteriostatic environment²

Branched chain fatty acids

forms an acidic semi-permeable layer around the wound that acts as a barrier against external contamination

4

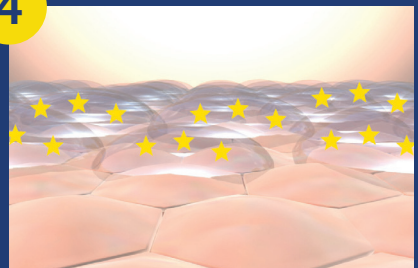


Contains Arginine²

Arginine

functions as a building block of the skin and supports optimal moisturisation

How to use?²

1

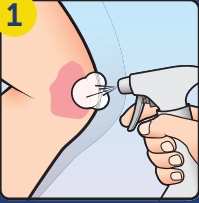

CLEAN

Clean the skin and wound with an irrigation solution


2



DRY

Dry the wound gently

3

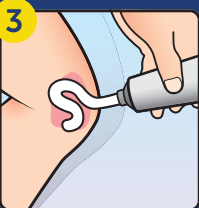

TREAT

Apply a layer of Flamigel® 1-2x/day
In case of sunburn: apply 2-6x/day

4



COVER

Cover the wound with a plaster or sterile non-adhesive dressing

Always read the label and follow the directions for use

Reference:

1. Thomas, S. Wound management and dressings. The pharmaceutical press London. 1990.
Chapter 2: Functions of a wound dressing, p9-19
2. Flamigel® Instructions for Use

® Registered Trademark. Flamigel® are the registered trademarks of Flen Health NV.



Order Code: 1005

CE 0344

Sentry Medical Pty Ltd

22 Peter Brock Drive, Eastern Creek, NSW 2766

1300 995 999 | sales@sentrymedical.com.au | www.sentrymedical.com.au

 **Sentry**
MEDICAL