

ISB301HT - IV Dressing Reinforced HT *(Instructions For Use)*

The Asguard Hi-Tac Dressing range delivers high initial tack with sustained, firm securement for reliable fixation. The dressing features high-tack laminated non-woven regions that enhances securement and helps maintain dressing stability throughout wear. The dressing adheres firmly while still allowing clear visibility of the insertion site for ongoing monitoring.



STEP 1

Prepare site as per local protocol and ensure skin is clean and dry before application.
Remove backing to uncover adhesive side of dressing.



STEP 2

Position clear window over insertion site, positioning the cut-out to enable access to IV cannula.
Apply gentle pressure across the surface of the dressing, ensuring adherence.



STEP 3

Position non-woven tabs under cannula and press to adhere, ensuring the hub is positioned above dressing.



STEP 4

Smooth edges of dressing whilst gently removing the paper frame.



STEP 5

Ensure thorough adhesion by smoothing down dressing from centre to outer edges.



STEP 6

Apply securement tape(s) as preferred and required by local protocol.



REMOVAL

To minimise potential damage to fragile skin, Asguard IV Dressing Reinforced Hi-Tac should be eased off using the "stretch and release" technique or with an adhesive remover wipe.

INDICATIONS

Asguard IV Dressing Reinforced Hi-Tac are used to dress IV insertion sites and to provide primary fixation of intravenous access devices.

CONTRAINDICATIONS

Should not be used if allergies to any of its components is known.

At Sentry, we are dedicated to upholding sustainable principles to safeguard our planet, people, and future, and we strive to consistently act responsibly in doing so.

Sentry Medical Pty Ltd

1300 995 999 | sales@sentrymedical.com.au
www.sentrymedical.com.au

