

ISB001HT, ISB002HT, ISB003HT & ISB004HT

Transparent Film Dressing HT (Instructions For Use)


The Asguard Transparent Film Dressing HT is a transparent polyurethane adhesive film with a high-tack lamination which adheres to the skin and forms a flexible and comfortable layer. Asguard Transparent Film Dressing HT conforms to the skin and allows easy visibility to wound healing. They are ideal for the management of all types of insertion sites to provide a waterproof environment and/or to keep primary dressings in place.

STEP 1




Insert IV cannula using local protocols. The skin surrounding the wound should be clean and dry.

STEP 2

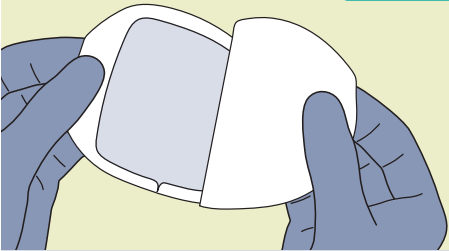


Open the sterile packaging and remove the dressing. Separate the paper backing from the Asguard Transparent Film Dressing HT.

STEP 3

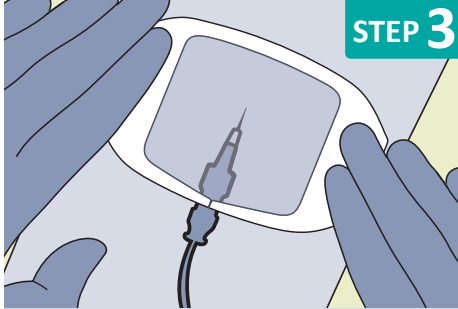

Apply the Asguard Transparent Film Dressing HT covering insertion site and IV cannula hub. Apply gentle pressure around the edge of dressing.

STEP 4

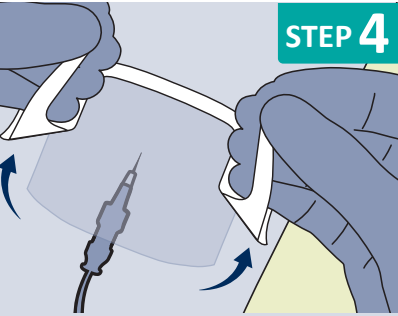

Peel the window frame away from the dressing. Smooth down the edges of the transparent border to create a waterproof and secure seal. Complete and apply dating label as preferred and required by local protocol.

DRESSING CHANGE



7 days is the maximum wear time for the dressing.

To minimise potential damage to fragile skin, Asguard Transparent Film Dressing HT should be eased off, using a "stretch and release" technique or an adhesive remover wipe.

INDICATIONS

Asguard Transparent Film Dressing HT are indicated for use as dressings in the management of all types of insertion sites such as peripheral IV cannula or used as a fixation layer to provide a waterproof environment and/or to keep non-adhesive primary dressings in place.

CONTRAINDICATIONS

Should not be used if allergies to any of its components, is known.

At Sentry, we are dedicated to upholding sustainable principles to safeguard our planet, people, and future, and we strive to consistently act responsibly in doing so.



Sentry Medical Pty Ltd
1300 995 999 | sales@sentrymedical.com.au
www.sentrymedical.com.au

